scata

Ⓛ Basal-Thermometer SC 2161

CE 0197

- Ⓜ Clinical Electronic Thermometer SC 2161
- Ⓧ Thermomètre basal numérique SC 2161
- Ⓝ Digitale basale thermometer SC 2161
- Ⓜ Termometro basale digitale SC 2161
- Ⓧ Termómetro basal digital SC 2161

Rev. 17/08/2021-D,GB,F,NL,I,E

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K-jump Health Co., Ltd.  
No.56, Wu Kung 5th Rd.  
New Taipei Industrial Park  
New Taipei City, 24890 Taiwan

Polygreen Germany GmbH  
Ruhlsdorfer Str. 95  
14532 Stahnsdorf, Germany

#### Importatore

SCALA Electronic GmbH  
Ruhlsdorfer Str. 95  
14532 Stahnsdorf  
www.scale-electronic.de

### Device description and intended use

The 2-in-1 thermometer is intended to measure women's body temperature and check crystallization pattern of saliva. It has precise digital display and magnifier. It can be used in a household environment and in clinic by a doctor as reference.

#### Special features

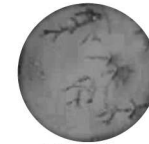
- Oral temperature measurement in around 30 seconds with proper use.
- A checker of crystallization pattern of saliva.
- 40 sets memories, recall all memories with NFC (Near Field Communication) mobile device.
- This device is not water proof.
- User-friendly APP BBT OUCare.

#### About basal body temperature


Basal body temperature is the lowest temperature attained by the body during sleep. It is generally measured immediately after awakening and before taking any physical activity, although the temperature measured at that time is somewhat higher than the true basal body temperature. For women, ovulation causes an increase of one-half to one degree Celsius in basal body temperature (BBT); monitoring of BBT is a way of estimating the day of ovulation.

#### About saliva pattern


Not only basal body temperature, but also saliva pattern is an indicator of whether ovulation or not. When your estrogen increases near your time of ovulation, your dried saliva will form a fern-shaped pattern.

Follicular Phase

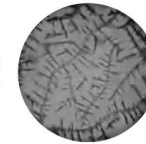Ovulation


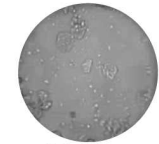
Luteal Phase

GB-1



### Parts description

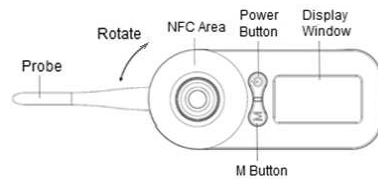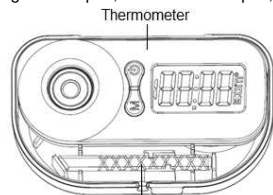[ Content ] Storage case 1pcs, Thermometer 1pcs, Saliva tool 1pcs, Adhesive tape 2pcs.

#### Front view

#### Back view
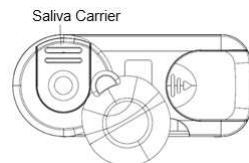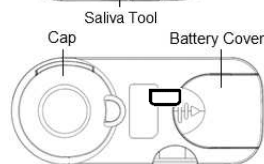
### Operating manual

1. Recommended to set the date and time of the thermometer when first use or after battery replacement.

2. Press the POWER button. The display will show: 

3. Release the POWER button and the display will show L °C with °C flashing.

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4. Place the probe in the appropriate position.
5. Once the degree sign °C on the display has stopped flashing, the measured temperature is showed on the LCD.
6. The unit will automatically turn off in 3 minutes (approx.). However, to prolong battery life, it is best to turn off the thermometer by pressing and releasing POWER button.

### Setting

The unit can be set date, time, clock mode on or off and clock alarm. The setting sequence is year, month, day, hour, minute, clock mode and clock alarm.

1. Under the off or clock mode, press and hold the M button until the display shows a blinking year and press the Power Button for adjustment.
2. After current year is selected, set the year by pressing the M button and switch to next adjustment.

3. Besides above setting, user can do these setting by NFC mobile.

### Special Features

#### Normal Temperature Alarm:

An alarm will sound when the peak temperature has registered and is ready-to-read. The "Normal Temperature" alarm (four quick beeps followed by a pause) will sound repeatedly for approximately 5 seconds.

#### Memory Recall:

When the unit is off or in clock mode, press the M button to view latest memory, each press can see previous memory. These memories show from the latest to the oldest one, and memory showing includes sequence and temperature.

### Measurement location

#### Oral Use

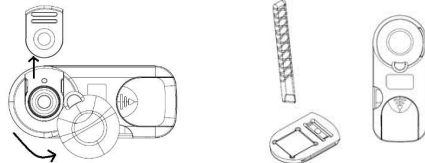
Place the probe well under the patient's tongue. Instruct the patient to keep their mouth closed while the thermometer is reading.

A normal temperature by this method is usually considered between 36.0°C to 37.5°C. The measuring period: approx. 30 seconds

#### Check the crystallization pattern of saliva


##### Saliva collection

1. Turn left to open the cap then slide the saliva carrier out.
2. Turn over the saliva carrier.
3. Take some saliva sample with the saliva tool, slide it on the lens, and then wait a moment to dry.

4. Put the saliva carrier back then close the cap.
5. Stick the double-adhesive tape on the NFC area of the thermometer, if not steady.
6. Turn on the LED by pressing and holding the Power Button for 3 seconds.
7. LCD shows "On", it means the LED lights, it is ready for checking the crystallization. The LED will be auto off after 180 seconds.
8. You can see the saliva pattern from the lens directly, or view it through the camera of a mobile device.

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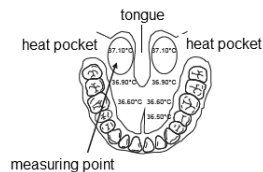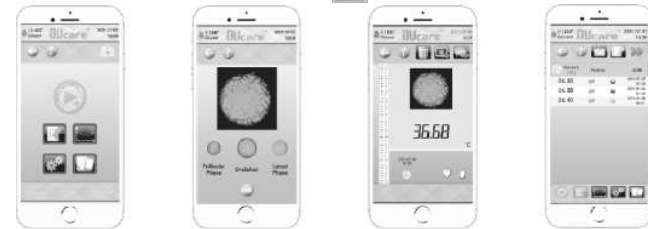
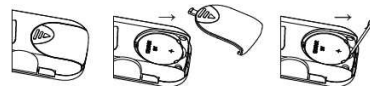
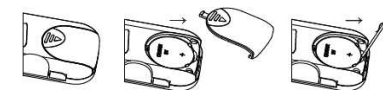
I-5

#### Changing the battery

1. When the symbol "  " appears, the battery needs to be replaced.
2. Remove the battery cover to insert a new battery (CR2032).
3. Confirm the polarity (+ and -) of the battery are correct as indicated inside the compartment and insert the battery accordingly.
4. Replace the battery cover.
5. After replacing the new battery, please set the thermometer again.

#### APP Operation

1. Install BBT OUcare APP successfully.

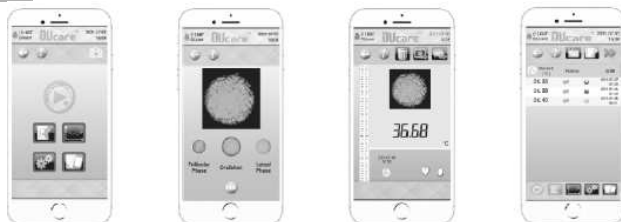
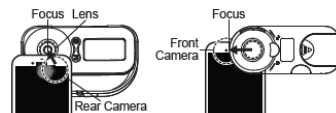
2. Execute BBT OUcare APP, focus the lens then stick the mobile on the device.
3. Press  to take a picture. Recommend to place the thermometer into the storage case for better angle when taking a picture of saliva crystallization.

GB-5

※ If the rear camera can't take a good picture, please use the front camera.

4. Compare the pattern and then select Follicular Phase, Ovulation or Luteal Phase then press **OK**.
5. The picture is ready and saved in the APP, you can press  to view the result.

**NOTE:**  means this reading has a saliva crystallization photo.

#### Cleaning and disinfection

Please only use an alcohol pad. Alcohol pads can be purchased from any dispensing chemist. We recommend the use of Mucocit®-F2000 or Helipur®Hplus to disinfect the thermometer when it is being used in medical facilities. Both are effective against all hospital-typical pathogens and can be purchased from any dispensing chemist.  
GB-6

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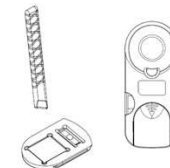
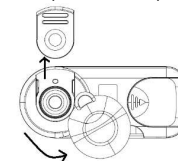
I-3

#### Safety information

- The measuring site is oral!
- The thermometer contains small parts (batteries, etc) which could be swallowed by small children. Please keep the thermometer and its batteries out of the reach of small children.
- Protect the SC 2161 from direct sunshine and temperatures above 50°C.
- Never drop the SC 2161 or open it (with the exception of the battery compartment for battery replacement).

#### Guarantee

We give a guarantee period of 2 years from the date of purchase. We guarantee perfect design of this article. We will rectify all material or manufacturing errors during the guarantee period. If, contrary to expectations, defects are found, please send the device back to importers address. Please enclose the proof of purchase and a description of the defect so that the problem can be rectified as soon as possible. The guarantee excludes damages caused by improper handling, related to the battery or to the packaging.

#### Disposal note

Defect devices or devices which are taken out of operation at the end of their service life may still contain residues which could present a risk to the environment. Please dispose of the device in a suitable and correct way at a collection point for old electrical items.

Batteries must be disposed of correctly. There are containers for battery disposal at battery dealers and in communal collection points.

### Specification

Measuring Range:	32.00~43.99°C Temperature below 32.00°C: display "L" for low. Temperature above 43.99°C: display "H" for high.
Resolution:	0.01 °C
Accuracy:	35.5°C ~ 42.0°C : ±0.1°C, Other : ±0.2°C
Memory:	40 sets memories
NFC:	All memory read-back iPhone7 and above with iOS 11.0 or above Android mobile device with NFC
Battery (included):	CR2032 *1pcs
Battery Life:	More than 1000 measurements
Dimension :	80mm x 34mm x 20mm
Weight:	Approx. 23 grams including battery
Beeper:	Approx. 5 sec. sound signal when peak temperature reached
Working Condition:	16°C ~ 40°C, R.H. of 15~95% non-condensing
Storage Condition:	-20°C ~ 50°C, R.H. of 15~95% non-condensing
Lens:	High Definition Optical Lens, M7 4P+IR650
•Aperture	F2.8
•Angle of shot	68
•Focus distance	Depends on mobile device
Certification:	ISO 13485

The manufacturer declare that the product is conform with the requirements of the regulations (EC) 1907/2006 (REACH) and 2011/65/EU (RoHS).

Specifications may change as technical improvements take place.

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