# D Kleintier-Thermometer SC 313 flex CE

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# GB Small Animal Thermometer SC 313 flex CE

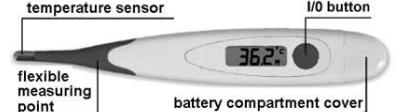
## Special features

The SC 313 is a maximum thermometer, i.e. the **highest temperature is displayed after approx. 10 seconds with proper use**. The most recent measured value is always saved.

## Safety information

- The SC 313 flex is suitable only for measuring animal body temperatures.
- The thermometer contains small parts (batteries, etc) which could be swallowed by small children. Please keep the thermometer and its batteries out of the reach of small children.
- Protect the SC 313 flex from temperatures above 50°C.
- Never drop the SC 313 flex or open it (with the exception of the battery compartment for battery replacement).

## Device description

## Operating manual

Press the I/O button to switch on. A short signal tone shows that the device is in the “switched on” condition.

**check all display segments ready for the measurement**

The current temperature is continuously displayed during measurement, and the „°C“ blinks during the entire measurement time. The measurement is completed after around 10 seconds with proper use.

After the measurement is completed, the temperature value measured remains in the display until the SC 313 flex is switched off. If the device is not switched off, a signal tone is sounded after around 3 minutes and the device then switches off automatically.

## Display of last measured value

If you keep the I/O button pressed for longer than three seconds, the last measured value will be shown in the display. As soon as you release the I/O button, the device is ready to take measurements.

## Cleaning and disinfection

The thermometer is waterproof. The thermometer is waterproof.

**Please note: Waterproof means not that the device is also waterproof in aggressive disinfection solutions and substances!**

Wipe the thermometer with a moist cloth which you have wetted with one of the following disinfection substances.

Disinfection substances	Manufacturer
Kodan® Tücher	Schülke & Mayr
EuroSept® Max Wipes	Henry Schein Inc.
Medizid Rapid	Rösner-Maut by Meditrade GmbH
Helipur® H plus N	B. Braun Melsungen

## Changing the battery

Pull the cover off the battery compartment. The battery can now be accessed. Remove the used battery with the help of a pointed object (pincers, small screwdriver etc.) and put the new battery (LR 41 or SR41) in. Please make sure it is the right way round.

## Disposal note

Defective devices or devices which are taken out of operation at the end of their service life may still contain residues which could present a risk to the environment. Please dispose of the device in a suitable and correct way at a collection point for old electrical items.

Batteries must be disposed of correctly. There are containers for battery disposal at battery dealers and in communal collection points.

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## Explanations of symbols

	Manufacturer
	Complies with EU directives
	Product type BF
	Disposal notes
	Production period (month/year)
	Read the instruction manual

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SCALA Electronic GmbH,