

scala®

Electronic Lice Comb SC 04



CE

GB Instruction manual

WARNINGS

Read the manual carefully to ensure safe and appropriate operation of your louse comb.

GENERAL NOTES

1. The Electronic Lice Comb SC 04 is not a medical device.
2. The Electronic Lice Comb SC 04 should be used on children by adults only.
3. The Electronic Lice Comb SC 04 is not water proof.
4. Do not hold metal combs with fingers when the POWER is on.
5. Some sensitive persons may feel slight tingling; there is NO electric shock risk.


6. The Electronic Lice Comb SC 04 should not be operated by or applied to persons with epileptic conditions and/or heart-failure patients.

7. The Electronic Lice Comb SC 04 may disrupt the operation of electronic health devices.

HOW DOES AN ELECTRONIC LICE COMB SC 04 WORK?

The Electronic Lice Comb SC 04 works by transmitting a small electrical charge to the metal comb. When the metal comb is combed through dry hair, this electric charge will kill adult lice. The Electronic Lice Comb's alert LED will show red light when a louse had been detected, killed and caught in the metal comb. Simply remove the louse with the mini brush. The Electronic Lice Comb SC 04 does not kill the eggs and some may remain attached to the hair follicle. It is recommended that combing treatments are repeated every 2-3 days for 16 days period to ensure that all lice been eliminated.

UNIT DESCRIPTION

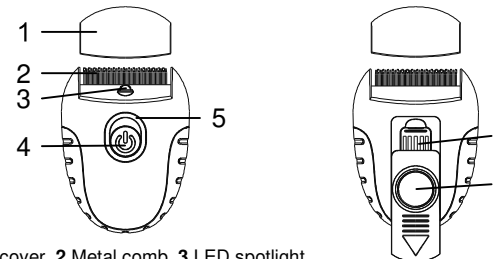

- 1 Head cover, 2 Metal comb, 3 LED spotlight,
4 Power switch, 5 Alert LED, 6 Mini brush, 7 Magnifier

GB-2

BUTTON FUNCTIONS

1st stage: Power Off

Push the power slide switch backward (1st position) to turn off the device.

2nd stage: Power On

Push the power slide switch forward (2nd position) to turn on the lice comb and the alert LED will show green light. If the battery power is insufficient, then the alert LED will show red light for five seconds and then turn off the power.

3rd stage: Power on & LED light on




Push the power slide switch forward (3rd position) turn on the lice comb and the illuminate the LED spotlight.

HOW TO USE


Before Use:

- Hair must be dry and clean before using this electronic device. For best results, hair should be free of styling product, i.e. gels or mousse.
- Before the treatment, comb the hair thoroughly with a regular comb to free all knots.

GB-3

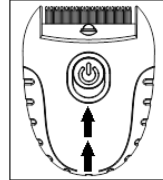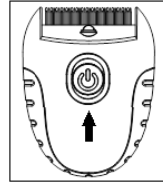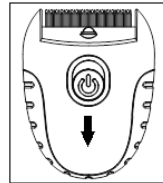

1. Remove the head cover. Press the power slide switch forward to 2nd stage to turn on the device and the alert LED will show green light. Press the power slide switch forward to 3rd stage to turn on the LED spotlight additional.

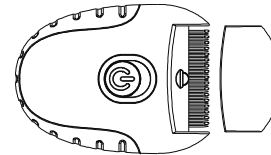

2. Hold the comb at a 45-degree angle (the ON button is facing up towards the user) and start combing with the metal comb near the hair-roots. The LED spotlight can help to see the louse in low light environment or on those persons with dark hair.
3. When the comb has trapped and killed a louse, the alert LED will show red light.
4. Stop combing the hair, and remove the louse from the metal comb. Use the mini brush to clean the metal comb. After the louse is cleared, the alert LED will show green light and the comb is ready for use again.



A demonstration how to use the lice comb you find at the internet: <https://youtu.be/XL5d8OEJL8k>

GB-4


ABOUT THE BATTERY

Warning:

1. The Electric Lice Comb with 1 AAA battery.
2. The batteries must be kept strictly out of the reach of children.
3. Always use new batteries of the type and specifications indicated in this manual.
4. Leaking batteries may damage the device. It is recommended to remove the battery whenever the device is not expected to be used for an extended period of time.
5. Always dispose of batteries in accordance with applicable legal regulations. Do not dispose of batteries with the normal household waste.

REPLACING THE BATTERY

To change the battery, slide and open the battery cover and remove battery. Replace the new battery being sure to align properly as indicated inside the battery compartment.

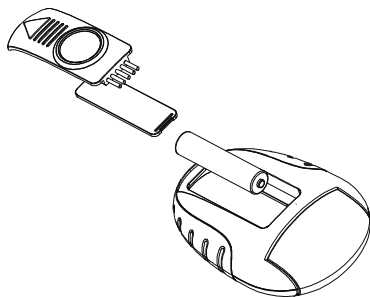


GB-5

CLEANING AND STORAGE

Cleaning:

- The metal comb can be cleaned with the included mini brush.
- Turn off the device and reverse the metal comb downward while you cleaning it. Then use the alcohol swab or the soft cotton moistened with the 70% alcohol to clean the metal comb.
- Keep the device dry and away from any liquids.

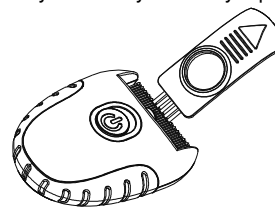

Storage:

Store the device in a secure, dry, dust-free place when not in use. Avoid extreme temperatures. Do not attempt to disassemble the device.

DISPOSAL NOTE



Defect devices or devices which are taken out of operation at the end of their service life may still contain residues which could present a risk to the environment. Please dispose of the device in a suitable and correct way at a collection point for old electrical items.









Batteries must be disposed of correctly. There are containers for battery disposal at battery dealers and in communal collection points.

GB-6

GUARANTEE

We give the legal guarantee period. We guarantee perfect design of this article. We will rectify all material or manufacturing errors during the guarantee period. If, contrary to expectations, defects are found, please send the device back to our customer service department. Please enclose the proof of purchase and a description of the defect so that the problem can be rectified as soon as possible. We don't give a guarantee in the case of a dirty device.

EXPLANATION OF SYMBOLS

	Manufacturer
	Conform EU rules
	Disposal notes
	Read the instruction manual
	Production period month/year
	Keep dry


TECHNICAL SPECIFICATIONS

Operating environment: 10°C - 40°C, with relative humidity: 25%-80%

Dimensions: 90 x 60 x 27 mm

Weight: 42 g (without battery)

Battery: 1 x AAA alkaline battery (LR03)


K-Jump Health Co. Ltd.
No. 56, Wu Kung 5th Rd.
New Taipei City 24890
Taiwan

importer / service
SCALA Electronic GmbH
Ruhlsdorfer Str. 95
14532 Stahnsdorf
Germany

GB-7