

# scala® „Never Forget“ Pill Box

- quartz timer,
- up to 5 alarms per day,
- audible, vibrating or audible + vibrating alarms,



GB Instructions for use



## The „Never Forget“ pill box with five alarm settings

We congratulate you on the purchase of this quality product. To be able to make best use of all features of this product you should read these instructions carefully before you use the pill box for the first time. Keep this instruction leaflet in a safe place so you can consult it if you need to.

### Features:

The pill box helps you organize your daily drug taking duties and reminds you that the time has come to take the next pill.

Other features of the pill box:

- Integrated quartz timer with large LC display (13 mm character height).
- Setting up to 5 alarm times a day. Once programmed, the alarm times are repeated daily.
- The alarm is audible, vibrating and audible or only vibrating.
- Blue lights make handling the pill box easy in darkness.

### Cautions / intended purpose:

- ◆ The pill box contains parts (batteries, etc.), which can be swallowed by small children and thus can be a direct danger. Therefore, store the pill box out of reach of children. If a child swallows a battery or other part, call a doctor immediately.
- ◆ Do not expose the pill box to direct sunlight or temperatures above 50°C, and protect it from water.
- ◆ Do not drop the pill box, and open it only for changing the battery or making a „Reset“.
- ◆ Please use the pill box only for pills. Any other purpose is forbidden by the manufacturer.

### Disposal note



Defect devices or devices which are taken out of operation at the end of their service life may still contain residues which could present a risk to the environment. Please dispose of the device in a suitable and correct way at a collection point for old electrical items.

Batteries must be disposed of correctly. There are containers for battery disposal at battery dealers and in communal collection points.

### The pill box




### Installing and changing batteries

The pill box is powered by two batteries. The CR 2025 or CR 2032 lithium battery powers the timer, an AAA size battery (LR03) is needed for the alarm functions (audible, vibrating or audible and vibrating) and the LCD light.

The lithium battery is already installed. The AAA battery must be installed in the battery compartment before the pill box can be used (mind correct polarity).

For changing the battery or installing the AAA battery, open the pill box by pushing the „OPEN ►“ slide at the side of the pill box in the direction of the arrow. This releases the lock between the top and bottom parts of the pill box and you can take the two apart. You find the battery compartment in the top half box. Make sure of the correct polarity of the batteries.

Change the battery when the LC display is getting dark (CR 2025 or CR 2032) or the lighting lacks brightness or the alarm volume (LR03).

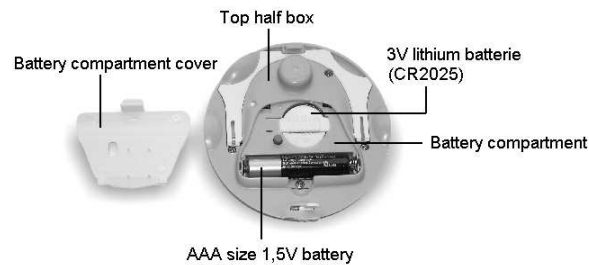

### Setting the time

1. Push the „HOLD“ slide at the side of the pill box to: HOLD ►
2. Press the „T-SET“ button below the LC display for about 3 seconds.  
On the LC display, you will see **T.SET** and the hours blink.
3. Set the hours by turning the time setting ring.
4. Press „T-SET“ again and the minutes blink. Set the minutes with the time setting ring.
5. Another press on „T-SET“ completes the time setting.  
**T.SET** disappears.

If you shift the „HOLD“ slide to HOLD ► all buttons are without function, i.e., the „HOLD“ switch locks the buttons and prevents accidental setting.

### Setting the 5 alarm times

1. Push the „HOLD“ switch at the side of the pill box to: HOLD ►
2. Press „AL-SET“ below the LC display for about 3 seconds.  
**ALSET** is displayed and the first alarm time „1 -:- -“ blinks.
3. Turn the time setting ring to set the first alarm time in 30-minute steps.
4. Press „AL-SET“ again to complete setting the first alarm time. The next alarm time „2 -:- -“ blinks.


You can set the second (third, fourth and fifth) alarm times with the time setting ring in the same way as the first alarm time or if you need one alarm time only press „AL-SET“ to advance to the fifth alarm time and complete the time setting by pressing „AL-SET“ once more.

**ALSET** disappears.

After you have completed the setting of alarm times, return the „HOLD“ switch to the position: HOLD ►.

### Resetting all times

Open the pill box by pushing „OPEN ►“ at the side of the pill box in the arrowed direction. Then press a toothpick or a similar tool in the „RESET“ reset. This resets all times. The timer shows 12:00 AM and the alarm times „-:- -“.

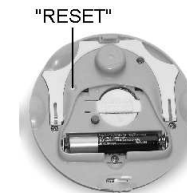



### Placing pills in the box

The pill box has five numbered compartments for pills on the reverse. Turn the transparent cover to fill the compartments or take out the pills.



Drug compartment cover

### LCD lighting

To switch on the LCD lighting for about 10 seconds, at first push the „HOLD“ slide to:  HOLD  and press the „Light“ button.

### Setting the alarm type

At the „BEEP VIB BEEP+VIB“ slide, you can choose between audible alarm, vibration alarm or combined audible and vibration alarm. This setting can be done without the „HOLD“ slide.



### The alarm / removal of drugs

If you note an alarm, it is time to take the drug from the compartment for which the alarm has been programmed by turning the transparent cover. Turning the cover silences the alarm and the alarm time in the LC display disappears. The display again shows „-:-“. The alarm is sounded or vibrated for about 30 seconds.

If you forget to take the drug from the compartment, the alarm is repeated twice at 5-minute intervals.

If you take the drug from the compartment up to 15 minutes before the set alarm time or turn the transparent cover, the alarm is not sounded.

If you forgot an alarm time and the drug is still in the compartment, the alarm time remains set in the LC display as a reminder.

Normally, you remove all drugs, and all alarm times in the LC display go to „-:-“.

At 12:13 AM every day the alarm settings are enabled again which ensures that the pill box uses the same time settings the next day again.

### Warranty

The „Never Forget“ pill box carries a warranty of two years. All defects are repaired free during the warranty.

No warranty is available for damage resulting from improper handling, accident, abuse or alterations by unauthorized individuals. The packaging, batteries and the operating instructions are also not covered by warranty.

### Technical data:

Operating voltage

Time:

3V (1 x CR 2025 or CR 2032)

Light and alarm functions:

1,5V AAA battery (LR03)

Quartz timer (accuracy +/- 20 sec./month)

optional: audible, vibrating or

Audible and vibrating

Timer

Alarm

Alarms/day

5

Drug compartments

5

Display

LC display with 13mm characters (time)

Dimensions

Ø 80 mm, height 31 mm

Weight

96 g (incl. batteries, excl. drugs)

LCD light

blue

### Importer

SCALA Electronic GmbH

Ruhlsdorfer Straße 95

14532 Stahnsdorf, Germany