# scala

## Electronic Body Scale

### SC 4120 user manual


Thank you for your choosing this electronic personal scale! To ensure proper operation and maintain the lifespan of the scale, please read the manual carefully before usage.

#### Features:

- ◆ Max Capacity: 180kg.
- ◆ Graduation: 50 gram.
- ◆ Weighing Range: 0.20kg - 180kg.
- ◆ Unit modes: kg/lb/st.
- ◆ High precision strain gauge sensor system.
- ◆ 6mm thick tempered glass platform.
- ◆ LCD display with backlight, Easy to read.
- ◆ Step-on Technology, Switch-on over 3kg.
- ◆ Auto-zero and Auto switch-off.
- ◆ Overload display indicator.
- ◆ Low battery display indicator.
- ◆ Battery Powered: 2 x AAA batteries.


#### Battery Powered:

Please put the batteries in the battery compartment at the back of the scale. And please make sure to match - and + signs on the batteries correctly when install the batteries.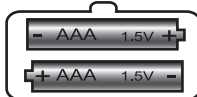

#### Units Switch:

The scale is designed with three kinds of weighing units to choose, kg/lb/st. Press the unit key at the back of scale to choose the weighing unit you'd like to use.

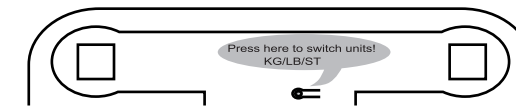
#### Weighing Procedures:

1. Keep the scale on the flat surface and stable floor, Let the scale shut off automatically.

2. Please step on the glass platform, the scale will turn on automatically. Stand still for some seconds until LCD display shows your body weight, the body weight is blinking.

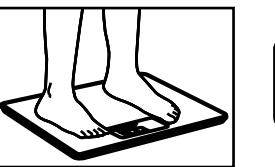3. The scale will turn off automatically after 5 seconds.

#### Special Indicators:

- Low Battery. Please remove the old batteries and change new ones.

- Overload. The maximum weight (180kg) had been exceeded.

#### Precautions:

- ◆ Please don't keep the scale on carpet or other soft floor to weigh in order to avoid inaccuracy.
- ◆ Please don't accumulate items on the surface of scale to avoid power consumption and shortening lifespan of battery when the scale is off duty.
- ◆ Please avoiding water splashing and keep the tempered glass platform to be dry in order to avoid slipping over when weighing and the damage of electronic parts of the scale.
- ◆ Please clean the scale tempered glass surface with a slightly damp soft cloth. Do not use corrosive cleaner or immerse the scale directly into water.
- ◆ Please pick up the battery when the scale won't be used for a long time.
- ◆ Please dispose the end-of-life batteries properly in order to protect our environment.
- ◆ The scale is designed only for household body weighing, not for commercial or medical use.

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#### GB Disposal notes

The defective device at the end of their life may contain material that might be a hazard to the environment. Dispose of the device as prescribed, i.e., at a collection point for electrical scrap. The disposal of batteries is regulated. Spent batteries are collected at collection points provided by battery dealers and the communities.

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K-jump Health Co., Ltd.,
SCALA Electronic GmbH