DE Haustier-Thermometer SC 1505 flex

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GB Pet Thermometer SC 1505 flex

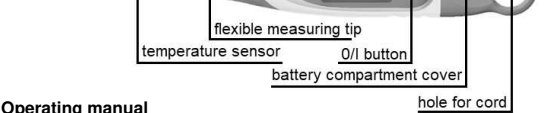
Special features

The SC 1505 flex is a maximum thermometer with a flexible measuring point, i.e. the **highest temperature is displayed after approx. 10 seconds with proper use**. The most recent measured value is always saved.

Safety information


- The SC 1505 flex is suitable only for measuring animal body temperatures.
- The thermometer contains small parts (batteries, etc) which could be swallowed by small children. Please keep the thermometer and its batteries out of the reach of small children.
- Protect the SC 1505 flex from temperatures above 50°C.
- Never drop the SC 1505 flex or open it (with the exception of the battery compartment for battery replacement).

Device description

Operating manual

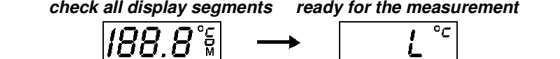Press the O/I button to switch on. A short signal tone shows that the device is in the "switched on" condition.

The current temperature is continuously displayed during measurement, and the „°C“ flashes during the entire measurement time. The measurement is completed after around 10 seconds with proper use. After the measurement is completed, the temperature value measured remains in the display until the SC 1505 flex is switched off. If the device is not switched off, a signal tone is sounded after around 3 minutes and the device then switches off automatically.

Display of last measured value

If you keep the O/I button pressed for longer than three seconds, the last measured value will be shown in the display. As soon as you release the I/O button, the device is ready to take measurements.

Cleaning and disinfection

The thermometer is waterproof.

Please note: Waterproof means not that the device is also waterproof in aggressive disinfection solutions and substances!
--

Wipe the thermometer with a moist cloth which you have wetted with one of the following disinfection substances.

Disinfection substances	Manufacturer
Kodan® Tücher	Schülke & Mayr
EuroSept® Max Wipes	Henry Schein Inc.
Medizid Rapid	Rösner-Maut by Meditrade GmbH
Helipur® H plus N	B. Braun Melsungen

Changing the battery

Unscrew the battery compartment cap and pull off the cap. The battery can now be accessed. Remove the used battery with the help of a pointed object (pincers, small screwdriver etc.) and put the new battery (LR 41 or SR41) in. Please make sure it is the right way round.

Disposal note

Defect devices or devices which are taken out of operation at the end of their service life may still contain residues which could present a risk to the environment. Please dispose of the device in a suitable and correct way at a collection point for old electrical items.

Batteries must be disposed of correctly. There are containers for battery disposal at battery dealers and in communal collection points.

Guarantee

We give a guarantee period of 2 years from the date of purchase. We guarantee perfect design of this article. We will rectify all material or manufacturing errors during the guarantee period. The guarantee excludes damages caused by improper handling, related to the battery or to the packaging.

Specifications

Type:	maximum thermometer
Measuring range:	32.0°C to 43.9°C Temperature below 32.0°C: display "L" for low. Temperature above 43.9°C: display "H" for high.
Measurement accuracy:	+/-0.1°C between 35.5°C and 42.0°C at a surrounding temperature of 18°C to 28°C. Automatic internal check of test value 37.0°C. In case of deviation > 0.1°C, "Err" will be displayed.
Self-test:	
Battery type:	1 no.; LR 41 or SR 41.
Battery display:	The " " shows when the battery is discharged.
Storage temperature:	-10°C to 50°C

Explanations of symbols

	Manufacturer
	Complies with EU directives
	Product type BF
	Disposal notes
	Read the instruction manual
	Production period (month/year)
	Compliant with EU directive 2002/95/EG

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SCALA Electronic GmbH, Ruhlsdorfer Straße 95, 14532 Stahnsdorf, Germany